

Instant Cold Packs

Instant Cooling | Portable Relief | First Aid Ready



Product Overview

The instant cold pack contains calcium ammonium nitrate and water, which are kept separate until you activate the pack. When you squeeze the pack, an internal barrier breaks, allowing the calcium ammonium nitrate to mix with the water. This mixing triggers an endothermic reaction, absorbing heat from the surroundings and causing a rapid drop in temperature. The pack quickly reaches its optimal cooling temperature, which can be as low as 39.2 degrees Fahrenheit, and provides about 20 minutes of cooling.

Features and Benefits

- Innovative metal disk mechanism guarantees instant fluid release
- Convenient fold over and squish activation for hassle-free use
- Ammonium nitrate technology for fast and reliable cooling
- Compact and portable design, perfect for on-the-go relief

Product Specifications

- Size: 6 x 9 inches
- Cooling Duration: Approximately 20 minutes
- Minimum Temperature: Does not drop below 50°F
- Formula Base: Calcium ammonium nitrate and water
- Activation Method: Fold and knead
- Application: Single-use cold therapy

Packaging Details

Item No.	Quantity	Size
SAM-1001	24/CS	6 x 9 in

Instructions



Squeeze



Shake



Wrap in Cloth



Apply



9 inches

6 inches